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INTENDED USE

Powder for the preparation of a washing solution for the cleaning and maintenance of US-3100 R plus™ and UC-3500 analytical systems.

METHOD OF USE

Completely dissolve the content of a packet in 2 liters of distilled water. For instruction for use follow the instruction manual of the instrument.

Concentration

Detergent Solution		
Component	Concentration	U.M.
Sodium Chloride	18	g

Conservation

Store at 15° - 30° C avoiding direct exposure to light.

Stability

The product, if stored at a temperature between +15° C and +30°C, is stable until the expiry date described on the outer label of the container. Once reconstituted the product is stable for 7 days.

PRECAUTIONS and WARNINGS

- Reagents and waste materials shall be disposed of in accordance with Community waste provisions or national or regional provisions.
- Reagents may contain non-active components such as preservatives and detergents. The total concentration of these components is below the limits set out in Regulation 1272/2008 EC and subsequent amendments and additions.

Reporting of serious incidents

In the event of a serious incident in relation to the use of the device, please inform the manufacturer (via your distributor) and the competent authority of the European Union member state where the incident occurred. For other jurisdictions, reporting must be made in accordance with regulatory requirements. Reporting serious incidents helps provide more information about the safety of the diagnostic medical device.

Symbols used in IFU and Packaging	
 In vitro diagnostic medical device vitro	 Manufacturer
 Catalogue Number	 Instruction for use
 Lot Number	 Temperature limitation
 Expiration date	

REVISION	DATE	CHANGE
Rev.E	06/2023	New Issue for IVDR Regulation (UE) 2017/746 compliance